

## Includes

- MDT
- Sorbothane Pads (2)
- Lock Ease
- MDT Tips (5)

- A. TIP-SS Single Small Tip (Yellow)
- B. TIP-S Single Tip (White)
- C. TIP-DL Double Tip Large (White)
- D. TIP-DS Double Tip Small (Green)
- E. TIP-DM Double Tip Medium (Green)



## Getting Started:

- Always plug any Pettibon electronics into a surge protector.
- The force of the adjustment may be set between the range of 1-200 PSI.
- Adjusting is made easy with the lightweight body and interchangeable tips.
- **Suggested Tip Use:**
  - A. **TIP-SS Single Small Tip (Yellow):** Infant, Child, Small Adult, Spinous, Transverse, Atlas
  - B. **TIP-S Single Tip (White):** Adult Spine, Spinous, Transvers, Atlas, TMJ, Sacro-Iliac
  - C. **TIP-DL Double Tip Large (White):** Large Adult, Double Transverse Contact, P-A Adjusting
  - D. **TIP-DS Double Tip Small (Green):** Small Adult, Child, Double Transverse Contact
  - E. **TIP-DM Double Tip Medium (Green):** Medium Adult, Double Transverse Contact

## How it works:

The MDT is an electric solenoid driven instrument that is lightweight and versatile. It is designed to complement all adjusting techniques. Based on research under video fluoroscopy, the multiple toggles are timed at 3.3 per second, making them faster than the muscle's ability to react. This is VITAL to an effective and painless adjustment for your patient. Further, it was found that seated adjusting using the MDT was more effective than when the patient was lying down during the adjustment.

## Product Maintenance:

ALWAYS check your product before use for signs of wear or damage like cracks, frayed cords, unusual sounds, loose parts, unusual debris, and make sure cooling vents are cleaned regularly.



## Warranty Information:

- Extends 6 months from date of purchase.
- Covers manufacturing defects in workmanship and material and covers the cost of labor and parts if repair is necessary.
- Repairs must be done by the Pettibon System.
- **Warranty is void if:** Instrument is used incorrectly, Instrument has been opened or tampered with, or instrument was purchased from an unauthorized reseller.
- The Pettibon System is not liable for any consequence of incidental damage caused by failure of unit or improper use.