



Traction Wall Mount: 2-Piece Installation Instructions

Getting Started:

1. Choose the area where you would like to have the wall mount, locate a stud in the wall.
2. The bottom of the wall mount should be about 4 feet from the floor to best accommodate all of the heights of your patients.
3. Connect the 2 piece wall mount into one piece, the large end of the keyhole in the mount should be closest to ceiling, the smaller end of the keyhole should be toward the floor.
4. Use the 3- three inch screws to mount the unit to the wall stud.
5. Make a knot in the rope around the removable knob.
6. Insert the knob into the large end of the keyhole, and slide it down until it is securely seated in the small end of the keyhole and locked in place.

Includes:

- 2 piece wall-mount
- 1 Knob
- (3) three inch screws

Tools Needed:

- Phillips head screw driver
- Stud finder



Warning: If the unit is used incorrectly the knob can lift out of the keyhole and cause injury. Please make sure all patients are trained on proper form and technique before beginning use.

MAINTENANCE: Pettibon System recommends replacing products as needed – especially if they are under continuous use. If wear is noticed, replace IMMEDIATELY. Clinic use: replace often for maximum benefits.

We recommend consulting with your physician before beginning use of this product. Stop use immediately if any pain is experienced. No part of this form may be reproduced in any form without written permission from The Pettibon System®. © 2016, 2021, The Pettibon System. The Pettibon System is a registered trademark. All rights reserved. (Rev. 2/2016 – FOR CTS-EN)